Cold & Flu Relief

100% NATURAL RELIEF FROM
NASAL SYMPTOMS OF THE COMMON COLD & FLU

HYPERTONIC SEA WATER SOLUTION

ALGOMER
COMPLEX + EUCALYPTUS,
SPEARMINT & THYME

EN Sinomarin® Plus Algae Cold & Flu Relief with Algomer complex is a 100% natural spray that offers relief from nasal symptoms associated with the common cold and flu. It combines the natural decongestant action (osmosis effect*) of hypertonic sea water (2.3% NaCl) with the beneficial properties of seaweed (algae) extracts** present in **Algomer complex**, in addition to the aromas of Eucalyptus, Spearmint and Thyme. **Sinomarin® Cold & Flu Relief** cleanses the nasal cavities helping to eliminate infectious agents (mechanical action).

The seawater used in **Sinomarin® Cold & Flu Relief** is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents, and provides sea water of outstanding quality, rich in the sea's mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

Algomer complex is a combination of 100% natural ingredients extracted from *Undaria pinnatifida* and *Spirulina platensis* algae, namely, phycocyanin and a series of polysaccharide complexes including fucoidans. *Undaria pinnatifida* and *Spirulina platensis* algae are harvested from the most pristine waters of Patagonia in Argentina and Brittany in France, respectively, and are certified organic.

The essential oils of **Eucalyptus globulus** & **Mentha spicata** and **Thymus vulgaris** extract are well-known for their fragrant qualities and their ability to provide a pleasant aromatic sensation. **Sinomarin® Cold & Flu Relief** also contains **Glycerin** and **Propanediol** used as solvents. All components are 100% natural.

Sinomarin® Cold & Flu Relief can be used alone, as a natural alternative to medicated decongestants (local corticosteroids, antihistamines, vasoconstrictors), or in combination with them to help enhance their efficacy.

Properties:

- ▶ Relieves symptoms of the common cold & flu including nasal congestion, sinus pressure and runny nose.
- ▶ Fluidifies mucus secretions.
- ▶ Cleanses the nasal cavities removing mucus and mucus-trapped infectious agents (via mechanical action).
- ▶ Moisturizes the nasal mucosa and relieves nasal dryness.
- ▶ Helps promote natural recovery of the nasal mucosa.

Sinomarin® Cold & Flu Relief is recommended:

- ▶ For the natural relief of nasal symptoms associated with the common cold & flu (e.g. nasal congestion, runny nose) and in cases of rhinosinusitis.
- ▶ To eliminate infectious agents that may lead to secondary infections e.g. sinusitis, otitis (via mechanical action).
- ▶ In cases where medicated decongestants are contraindicated (e.g. high blood pressure, heart conditions, diabetes).
- ▶ To help enhance the efficacy of medicated decongestants.

Composition: Sea water (equivalent to 2.3% NaCl). Purified water, Algomer complex (*Undaria pinnatifida* & *Spirulina platensis* extracts), Aromas (Eucalyptus *globulus* essential oil, *Mentha spicata* essential oil, *Thymus vulgaris* extract), Glycerin of vegetal origin, Propanediol.

Sinomarin® Cold & Flu Relief is free of drugs, preservatives and chemical additives. 100% natural.

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ALGOMER
COMPLEX + EUCALYPTUS,
SPEARMINT & THYME

Recommended dosage:

For children over 6 years and adults. Use as per below, or as per physician's instructions. May be used without interruption for up to 30 days.

6-12 years: 2 to 3 sprays in each nostril up to 3 times daily.

>12 years: 2 to 4 sprays in each nostril up to 5 times daily.

Precautions: For nasal use only. Do not use on more than one individual. Do not use past expiry date. Keep out of reach of children. Do not use during pregnancy or breastfeeding. Do not use in cases of sensitivity or if allergic to any of the components.

Advice: A mild and temporary irritation may be experienced at the beginning of use.

Storage conditions:

Store below 25°C. Protect from sunlight.

Other information:

Special valves prevent contamination of the solution once the spray has been opened and thereby the necessity to use preservatives.

Sinomarin® is a complete line of hypertonic sea water solutions covering the needs of all ages from newborns to adults.

Sinomarin® products are CE-marked Medical Devices conforming to the requirements of Directive 93/42 of the European Union.

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Instructions for use / Instructions d'utilisation / Инструкции за употреба

- Blow nose to clear out mucus before using Sinomarin® Cold & Flu Relief.
- Remove protective cap.
- Hold the spray bottle upright and spray a couple of times in order to activate the spray pump.
- Bend neck gently forward over a wash basin and tilt head to the right.
- Gently insert applicator into the left nostril, in parallel with the nasal septum, and press firmly.
- Turn head to opposite side and repeat application in other nostril.
- Remove spray from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- Repeat procedure if necessary.
- Wash applicator well with warm water and wipe dry after each use.

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* Natural process characterized by water movement out of the swollen nasal tissues due to the local difference in salt (NaCl) content induced by nasal application of Sinomarin®.

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Sinomarin® Plus Algae Cold & Flu Relief is covered by granted international patent applications awarded to Gerolymatos International S.A., E.U.

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Sinomarin® is a registered trademark for Gerolymatos International S.A., E.U.
Sinomarin® est une marque déposée par Gerolymatos International S.A., U.E.