ISOTONIC

0.9% NaCl

EN Sinomarin® Isotonic Nasal Spray is an isotonic sea water solution (0.9% NaCl) naturally abundant in the sea's mineral salts and trace elements. It cleanses the nasal cavities by washing-out excess mucus and mucus-trapped impurities such as pathogens, allergens and pollutants, helping to keep the nose healthy and prevent secondary E.N.T. infections. Sinomarin® Isotonic Nasal Spray helps relieve nasal symptoms such as obstruction and mucus discharge caused by e.g. the common cold and allergies, helping to improve nasal breathing. It also moistens the nasal mucosa in dry conditions.

Indications

- For daily hygiene of the nasal cavities to remove mucus, pathogens (e.g. viruses, bacteria), allergens (e.g. pollen, house allergens), pollutants (e.g. air pollution, tobacco), dust and other potentially harmful particles (mechanical action).
- To help relieve nasal symptoms (e.g. obstruction, mucus discharge) caused by e.g. the common cold and allergies, rhinosinusitis, or following endonasal surgery.
- To moisten the nasal mucosa relieving dryness.

Sinomarin® Isotonic Nasal Spray can also be used in combination with medicated topical decongestants, preferably immediately prior to their application. Suitable during pregnancy and breastfeeding.

Composition

Sea water 28%, purified water 72% (1 ml of solution contains 0.28 ml of sea water), equivalent to 0.9% Sodium Chloride. Free from preservatives or other chemical additives. All natural. 100 ml / 125 ml: Include inert propellant gas (nitrogen), not in contact with the solution.

Description

Sinomarin® Adults Isotonic: 125 ml nasal spray. Normal spray suitable for adults and children over 6 years.

Sinomarin® Children Isotonic: 100 ml nasal spray. Gentle mist spray with safety tip, suitable for children and babies (0+).

Recommended dosage

Sinomarin® Adults Isotonic: 1-3 sprays in each nostril, 2-4 times daily.
Sinomarin® Children Isotonic: 1-2 sprays in each nostril, 2-3 times daily.

Adverse reactions

A stinging feeling or transitory irritation may be experienced at the beginning.

Precautions

For nasal use only. Do not use past expiry date. Keep out of children's reach. Do not use in case of hypersensitivity to sea water.

Storage conditions

Store below 25 °C. Pressurized container. Protect from sunlight and sources of heat. Do not expose to temperatures above 50 °C. Do not puncture or incinerate container even when empty.

Sinomarin® products are medical devices conforming to the requirements of Directive 93/42 of the European Community.

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Instructions for use / Instructions d'utilisation / Инструкции за употреба

Adults & Children

- 1 Blow nose to clear out mucus before using Sinomarin®.
- 2 Break tamper-proof seal on bottle and remove protective overcap.
- 3 Place nozzle on top of the bottle.
- 4 Bend neck gently forward over a wash basin and tilt head to the right.
- 5 Gently insert nozzle into the left nostril, in parallel with the nasal septum, and press firmly.
- 6 Turn head to opposite side and repeat application in the other nostril.
- 7 Remove nozzle from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- 8 Repeat procedure if necessary.
- 9 Remove nozzle from bottle wash well with warm water and wipe dry after each use.

Babies & toddlers who require assistance

- 1 Wipe baby's nose clean before using Sinomarin®.
- 2 Break tamper-proof seal on bottle and remove protective overcap.
- 3 Place nozzle on top of the bottle.
- 4 Place baby in supine position turning its head to the right.
- 5 Gently insert nozzle into the left nostril, in parallel with the nasal septum, and press firmly.
- 6 Turn head to opposite side and repeat application in the other nostril.
- 7 Remove nozzle from baby's nostril, return baby to upright position while allowing solution to work for a few seconds and then wipe the nose clean.
- 8 Repeat procedure if necessary.
- 9 Remove nozzle from bottle, wash well with warm water and wipe dry after each use.

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Sinomarin® is a registered trademark for Gerolymatos International S.A., E.U.
Sinomarin® est une marque déposée par Gerolymatos International S.A., U.E.
Gerolymatos International S.A.,